Welch Allyn Connex® Office profile
Addendum, software versions 1.0X–1.7X
Addenda, Versions du logiciel 1.0X–1.7X
Adición, Versiones de software 1.0X–1.7X

WelchAllyn®

Advancing Frontline Care™

© 2012 Welch Allyn. All rights are reserved. To support the intended use of the product described in this publication, the purchaser of the product is permitted to copy this publication, for internal distribution only, from the media provided by Welch Allyn. No other use, reproduction, or distribution of this publication, or any part of it, is permitted without written permission from Welch Allyn. Welch Allyn assumes no responsibility for any injury to anyone, or for any illegal or improper use of the product, that may result from failure to use this product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual.

Welch Allyn and Connex are registered trademarks of Welch Allyn.

Vital Signs Monitor 6000 Series is a trademark of Welch Allyn.

SpHb and Masimo are registered trademarks of Masimo Corporation. Possession or purchase of a Masimo SpO2- or Masimo SpHb-equipped device does not convey any express or implied license to use the device with unauthorized sensors or cables which would, alone or in combination with this device, fall within the scope of one or more of the patents relating to this device.

Braun is a registered trademark of Braun GmbH.

Software in this product is Copyright 2012 Welch Allyn or its vendors. All rights are reserved. The software is protected by United States of America copyright laws and international treaty provisions applicable worldwide. Under such laws, the licensee is entitled to use the copy of the software incorporated with this instrument as intended in the operation of the product in which it is embedded. The software may not be copied, decompiled, reverse-engineered, disassembled, or otherwise reduced to human-perceivable form. This is not a sale of the software or any copy of the software; all right, title, and ownership of the software remain with Welch Allyn or its vendors.

For information about any Welch Allyn product, call Welch Allyn Technical Support:

USA +1 800 535 6663	Australia +61 2 9638 3000
+1 315 685 4560	
Canada +1 800 561 8797	China +86 21 6327 9631
European Call Center +353 46 90 67790	France +33 155 69 58 49
Germany +49 695 098 5132	Italy +39 026 968 2425
Japan +81 42 703 6084	Latin America +1 305 669 9003
Malaysia +603 7875 3341	Netherlands +31 202 061 360
Singapore +65 6419 8100	South Africa +27 11 777 7555
Spain +34 917 499 357	Sweden +46 85 853 6551
United Kingdom +44 207 365 6780	

REF 105010

Material Number 718680, 80017674 Ver. B

Software versions 1.0X–1.7X

Welch Allyn, Inc.
4341 State Street Road
Skaneateles Falls, NY 13153-0220 USA

www.welchallyn.com

EC **REP**

Regulatory Affairs Representative
Welch Allyn Limited
Navan Business Park
Dublin Road
Navan, County Meath
Republic of Ireland

Advancing Frontline Care™

Contents

Welch Allyn Connex® Office profile	1
Introduction	1
Office profile	1
Approved accessories	12
Profil Consultation de Welch Allyn Connex®	15
Introduction	15
Profil Consultation	15
Accessoires certifiés	26
Perfil de oficina Connex® de Welch Allyn	29
Introducción	29
Perfil de oficina	29
Accesorios aprobados	41

Welch Allyn Connex® Office profile

Introduction

This addendum supplements the following documents:

- *Welch Allyn Connex® Vital Signs Monitor 6000 Series™ directions for use* (part number 103730)
- *Welch Allyn Connex® Integrated Wall System directions for use* (part number 104066)

The addendum describes capabilities of the Office profile that are not covered in the directions for use. Before using the product, you must familiarize yourself with the sections of the directions for use and this addendum that pertain to your use of the product.


Office profile

The Office profile is designed for outpatient clinical settings, such as a physician's office.

The profile supports two features that are not available in other profiles:

- Noninvasive blood pressure (NIBP) averaging programs: These programs display the average of multiple NIBP readings.
- Calculation of body mass index (BMI): The profile calculates BMI based on weight and height input.

You can also take manual NIBP and temperature measurements, monitor pulse rate and SpO2, and manually enter pain level.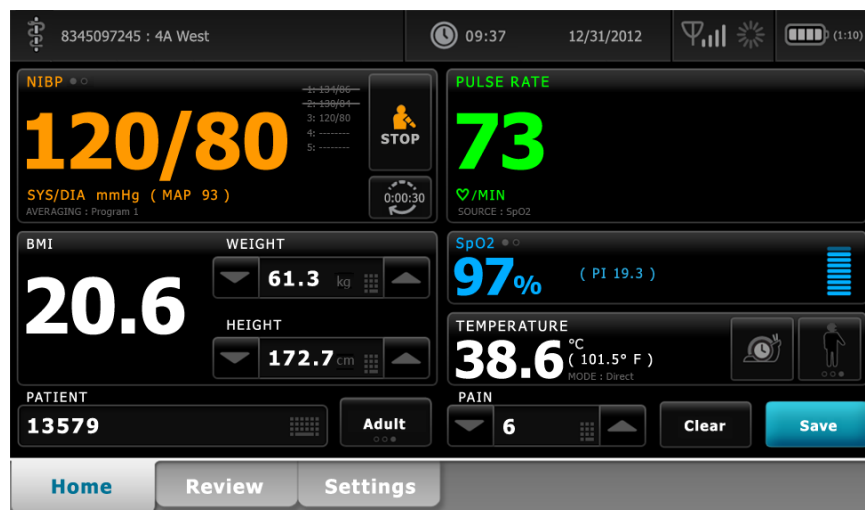

Profile feature comparison

The following table compares the features of the profiles.

Feature	Monitor	Spot Check	Triage	Office
Take NIBP, SpO ₂ , temperature, and pulse rate readings	X	X	X	X
Take SpHb readings (Masimo only)	X			
Configure and use interval timing setting	X			
Configure and use NIBP averaging programs				X
Observe and configure alarm limits	X			
Observe and respond to physiological alarms	X			
Change patient type (adult, pediatric, neonate)	X	X	X	X
View and enter manual parameters				
Temperature ¹	X	X		
Weight ²	X	X		X
Height ²	X	X		X
BMI ³	X	X		X
Pain	X	X		X
Respiration rate	X	X		
Save currently displayed data to device memory	X	X	X	X
Save and review patient data	X	X	X	X
Clear patient data without saving				X

Feature	Monitor	Spot Check	Triage	Office
Access Patients tab	X	X		
Access Alarms tab	X			
Access Review tab	X	X	X	X
Access Settings tab	X	X	X	X

¹ Braun IR thermometers configured to work with the monitor transfer temperature data automatically to the Temperature frame. You can enter temperature manually if you take a patient temperature with a thermometer that is not connected to the monitor, and you have selected temperature as one of the four manual parameters to display.


² These profiles support manual entry of weight and height measurements. If an approved weight scale is attached to the monitor, these profiles also display weight and height measurements taken by the weight scale.

³ In the Monitor and Spot Check profiles, body mass index (BMI) is calculated and transferred to the monitor only by an attached weight scale. You cannot enter or adjust BMI values. BMI appears on the Manual tab and in the Manual parameters frame if you have selected it as one of the four parameters to display.

The Office profile calculates BMI based on weight and height measurements (see Note 2). You cannot enter or adjust BMI values.

Navigation

Content area

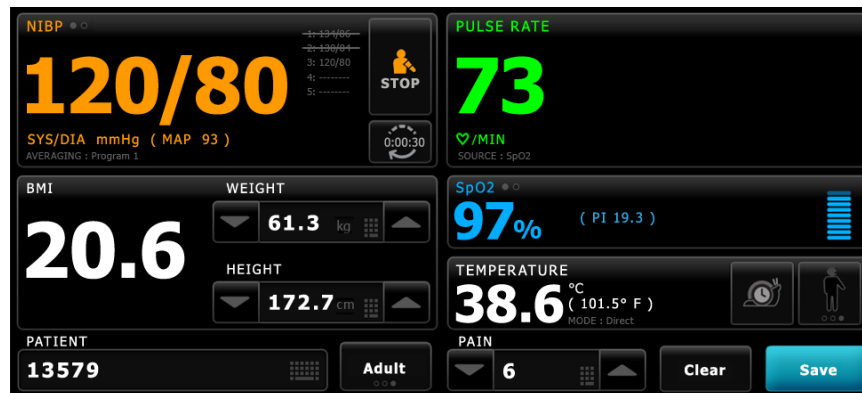The Content area displays vital sign measurements. It also provides shortcuts to several controls.

The Content area of the Office profile includes the following frames:

- NIBP
- BMI (including weight and height)
- Patient
- Pulse rate
- SpO2
- Temperature
- Pain

The Content area also includes a **Save** button, which you use to manually save current measurements.

The **Clear** button deletes current measurements without saving them.

Save patient data

Patient data can be saved to the monitor.

After taking a patient reading, touch **Save**.

A message appears indicating a successful save or a failed save.

Note During an NIBP averaging program, after acquiring an NIBP measurement, the monitor saves all measurements displayed on the Home tab and associated patient identification. To retain an NIBP average, touch **Save** upon completion of the averaging program.

Alarm messages and priorities

The Office profile displays technical alarms. It does not display physiological alarms.

Technical alarms

In addition to the technical alarm messages listed in the directions for use, the Office profile displays the following:

Alarm messages	Priority
Time limit exceeded. Unable to complete program.	Low

Patient data management

The Office profile supports manual entry of patient information.

Some configurations also enable scanning of patient barcodes. These configurations, specified in Advanced Settings, include the following:

- The "Primary label" setting is Patient ID.
- The "Primary label" setting is Name and the "Search by patient ID" setting is selected.

Manually enter patient information

In the Office profile, you can manually enter patient information in the Patient frame.

1. From the Home tab, touch the keyboard icon in the Patient frame.

The keyboard appears.

2. Enter patient information. Touch  , if displayed, to cycle through the patient data fields.

Note The "Primary label" setting in Advanced Settings specifies available fields.

3. Touch **OK**.
The information appears in the Patient frame.
4. To change the patient type, touch the patient type button (located on the right side of the Patient frame).

CAUTION Verify patient identity on the monitor after manual or barcode entry and before printing or transferring patient records.


Patient monitoring

Noninvasive Blood Pressure (NIBP) frame

From the NIBP frame, you can measure blood pressure.

Located in the upper left corner of the Home tab, the NIBP frame contains data and features relevant to noninvasive blood pressure measurement. The frame provides different features based on the profile you are using.

NIBP frame in the Office profile

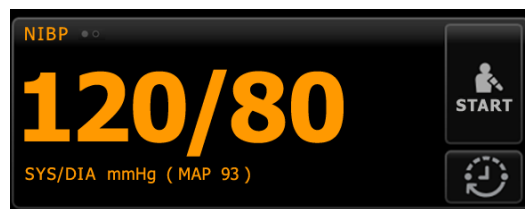NIBP measurement display

The frame can display systolic and diastolic measurements, and MAP calculations. You can configure the default view in Advanced settings.

View indicator

Touch the NIBP frame to toggle between views.

- NIBP view 1  displays the SYS/DIA measurements as the primary content and the MAP calculation as secondary content.
- NIBP view 2  displays the MAP calculation as the primary content and the SYS/DIA as secondary content.

Buttons

Button name	Button image	Description
Start/Stop		Touch to start a manual measurement.

Button name	Button image	Description
		<ul style="list-style-type: none"> During a manual NIBP measurement, touch to stop the measurement. During an NIBP averaging program, touch to stop the current measurement and terminate the program.
Interval		Touch to access the Intervals tab, where you can start an NIBP averaging program.
		<ul style="list-style-type: none"> Counts down to the next measurement during an NIBP averaging program. Touch to access the Intervals tab, where you can stop the NIBP averaging program.

NIBP measurement

The Office profile enables you to take manual NIBP measurements and use NIBP averaging programs.


Before taking any NIBP measurements in this profile, you must understand the following sections of the directions for use:

- "Select a cuff"
- "Position the cuff"
- "NIBP measurement"
- "Take a manual NIBP measurement"

NIBP averaging programs

An NIBP averaging program displays the average of multiple NIBP readings.


To calculate the average, the program takes a set of readings. The example below shows a program in progress: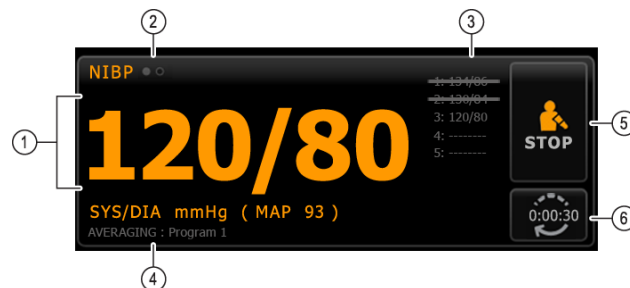

Number	Feature	Description
1	Numeric	Displays the most recent reading.
2	View indicator	Touch to toggle between NIBP views.

Number	Feature	Description
3	History	<ul style="list-style-type: none"> Displays completed readings and placeholders for upcoming readings. A reading with a line through it is excluded from the average.
4	Program	Displays the name of the program.
5	Stop	Touch to stop the current reading and terminate the program.
6	Interval	Counts down to the next reading.

After the program takes all readings, it displays the average reading, as follows: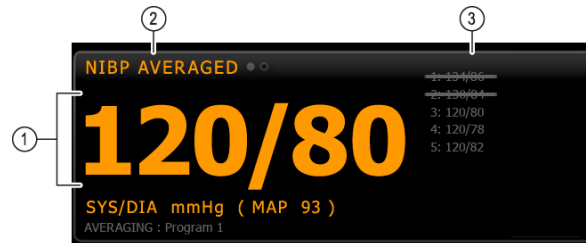

Number	Feature	Description
1	Numeric	Displays the average of the readings.
2	View indicator	Displays "NIBP AVERAGED."
3	History	<ul style="list-style-type: none"> Displays the readings taken by the program. A reading with a line through it is excluded from the average.

Set up an NIBP averaging program

In Advanced Settings, set up NIBP averaging programs to make them available for use.


To set up a program:

1. Access the Advanced Settings.
 - a. Touch the **Settings** tab.
 - b. Touch the **Advanced** tab.
 - c. Enter the **Advanced settings code**.
 - d. Touch **OK**.

The General tab appears.

2. Touch the **Parameters** tab.
3. Touch the **Programs** tab.
4. Touch the button of the program you want to set up.

The keyboard icon appears in the button.

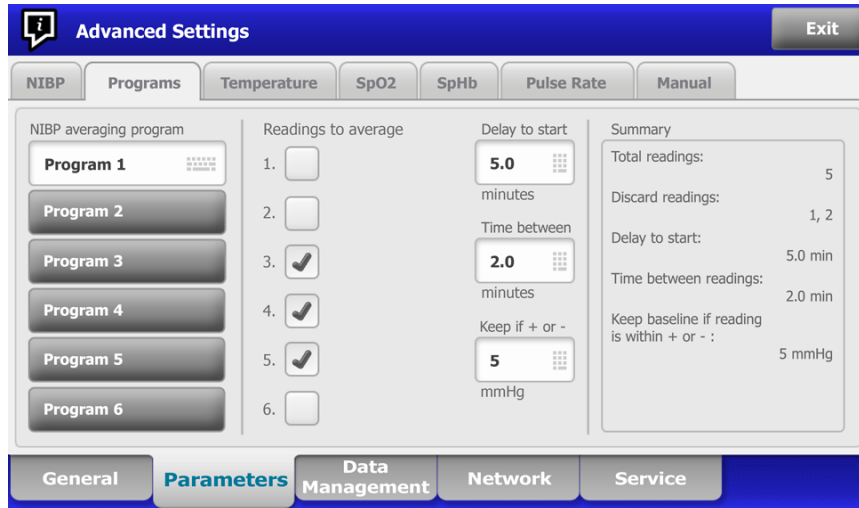5. (Optional) Change the name of the program:
 - a. Touch the keyboard icon.
 - b. Enter the name of the program and touch **OK**.
6. Specify the settings for this program.

Setting

Readings to average

Delay to start

Time between

Keep if + or -

Action/Description

Select readings to include in the average. You must select at least two readings.

The last reading you select is the final reading of the program. For example, if you select Readings 3, 4, and 5, the program takes five readings.

Unselected readings—in this example, Readings 1 and 2—are excluded ("discarded") from the average.

Enter the period between the start of the program (the moment that the "Start intervals" button is selected) and the start of the first reading.

Enter the period between the end of one reading and the start of the next reading.

Enter the range that the program uses to establish the baseline reading. For more information about how this setting affects the program, refer to the section "Excluded readings" below.

The settings you selected appear in the Summary area.

7. Do one of the following:
 - To continue in the Advanced Settings, touch another tab.
 - To exit the Advanced Settings and return to the Home tab, touch **Exit**.

The program is now available for use.


Start an NIBP averaging program

To acquire an NIBP average, start an NIBP averaging program from the Intervals tab.

1. Properly size the blood pressure cuff and position it around the patient's bare upper arm.

2. On the Home tab, touch .
- The Intervals tab appears.
3. Confirm that **NIBP averaging** is selected.
4. Touch the desired program.

The specifications of the program appear in the Summary area.

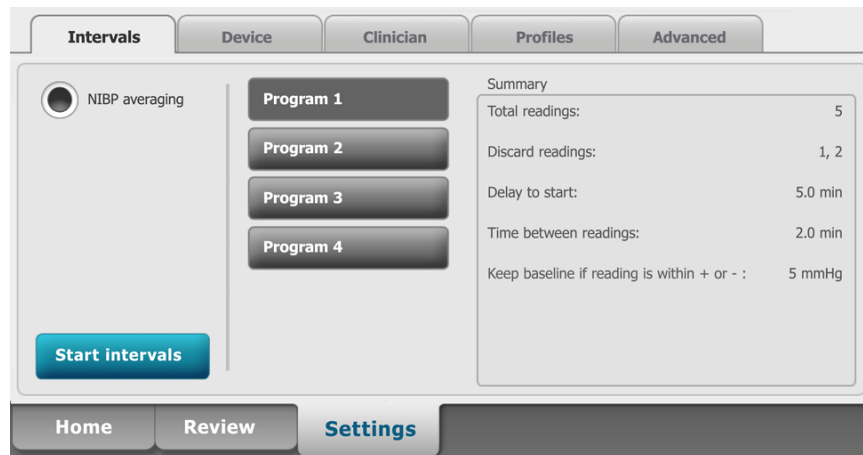5. Read the Summary information to confirm that the specifications are appropriate for your patient:

Setting

Total readings

Discard readings

Delay to start

Time between readings

Keep baseline if reading is within + or -

Action/Description

Number of readings that the program takes.

Readings that the program excludes from the average. For example, "1, 2" indicates that the program excludes the first and second readings.

Period between the start of the program (the moment that the "Start intervals" button is selected) and the start of the first reading.

Period between the end of one reading and the start of the next reading.

Range that the program uses to establish the baseline reading. For more information about how this setting affects the program, refer to the section "Excluded readings" below.

6. Touch **Start intervals** to start the program.

The Home tab appears.

In the NIBP frame, the timer counts down the "Delay to start" period. The first NIBP reading begins when the timer reaches 0.

After the first reading is acquired, the timer counts down the "Time between readings" period. The next NIBP reading begins when the timer reaches 0.

After taking all readings, the program displays the average.

Note

If a technical alarm condition occurs while a reading is in progress, the reading stops. The timer counts down the "Time between readings" period. When the timer reaches 0, the program attempts to take the reading again.

Note During an NIBP averaging program, the monitor saves all NIBP readings except the average. To retain an NIBP average, touch **Save** upon completion of the averaging program.

Stop an NIBP averaging program

To stop an NIBP averaging program in progress, touch  on the Home tab. The current reading stops, and the program terminates.

Note When a program has been stopped, it cannot be restarted at the point that it was stopped. To start a new NIBP averaging program, select a program in the Intervals tab and touch **Start intervals**.

Excluded readings

An NIBP averaging program excludes readings for the following reasons:

- The reading is listed in the "Discard readings" setting of the program.
- The reading precedes the baseline reading.

The program establishes the baseline reading as follows:

- At the start of the program, Reading 1 is the baseline reading.
- The program compares the systolic value of Reading 2 to the systolic value of Reading 1.
- If the difference between the values is within the "Keep baseline" range, Reading 1 remains the baseline. The program compares the next reading to Reading 1, and so on.
- If a reading is outside the range, that reading becomes the new baseline, and the program excludes from the average all readings that preceded the new baseline.
- After establishing a new baseline, the program compares subsequent readings to the new baseline and applies the rules described above.

BMI frame

The BMI frame displays body mass index (BMI), weight, and height.

Note This frame is available only in the Office profile.

Weight and height measurements can be entered manually or transferred from an attached weight scale. The profile calculates BMI based on weight and height input.

Note When a weight or height measurement is transferred from an attached weight scale to the monitor, the measurement displayed on the monitor is within one decimal place (0.1) of the measurement displayed by the weight scale.

Enter weight and height

The BMI frame enables you to enter weight and height measurements taken manually, and it displays weight and height measurements taken by an attached weight scale.

CAUTION Weight scales attached to this monitor must be running on battery power (battery type is specified in the weight scale manufacturer's directions for use). Do not use the weight scale's external power supply.

1. Verify that you are using the Office profile.
2. From the Home tab, touch the up/down arrow keys or the keypad to manually adjust weight and height.

Note If an approved, battery-powered weight scale is attached to the monitor, weight and height measurements from the weight scale populate fields in the BMI frame.

The BMI value changes based on weight and height input.

Pain frame

In the Pain frame, you can manually enter the patient's pain level.

Note This frame is available only in the Office profile.

Enter pain level

1. Verify that you are using the Office profile.
2. From the Home tab, touch the up/down arrow keys or the keypad to manually adjust the pain level.

Troubleshooting

This section supplements the troubleshooting information in the directions for use.

NIBP messages

Message	Possible cause	Suggested action
Time limit exceeded. Unable to complete program.	The averaging program could not be completed within the system time limit.	Check connections; limit patient movement. Clear the alarm and retry the program.

Problems and solutions

The problems addressed in this table do not generate alarm or information messages on the monitor.

Problem	Possible cause	Suggested action
No BMI value is displayed in the Office profile	<ul style="list-style-type: none"> The weight measurement is out of range The height measurement is out of range 	<p>Manually adjust the measurement.</p> <hr/> <p>Use the attached weight scale to retake the measurement.</p>
The Office profile does not appear in the Profiles tab	The Office profile license is not installed	Purchase a license and install the license using the service tool.

Approved accessories

Connex Vital Signs Monitor

Part Number	Description
401393*	Office Cart (requires Monitor Mounting Kit, PN 105065)
105065*	Monitor Mounting Kit (requires Office Cart, PN 401393)
105010	Addendum to the Directions for Use, Office Profile, Multilingual
105064	Quick Reference Card, Office Profile, English
105280	Quick Reference Card, Office Profile, French
105281	Quick Reference Card, Office Profile, Spanish

* Available for sale in the USA only. Part numbers 401393 and 105065 must be purchased and assembled together.

WARNING Do not attempt to move the Office cart system by pushing the post mount (marked with these symbols). The system can tip over when you push the post mount.

Connex Integrated Wall System

Part Number	Description
105010	Addendum to the Directions for Use, Office Profile, Multilingual
105064	Quick Reference Card, Office Profile, English
105280	Quick Reference Card, Office Profile, French

Part Number	Description
105281	Quick Reference Card, Office Profile, Spanish

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